
ResistancePlus™ MG Positive Control

95001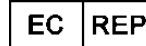

MT Promedt Consulting GmbH, Altenhofstrasse 80,
66386 St. Ingbert, Germany
Tel: +49 6894 581020, Email: info@mt-procons.com

SpeedX Pty Ltd, Suite G16 National Innovation Centre
Australian Technology Park, 4 Cornwallis Street, Eveleigh,
Sydney, NSW 2015, Australia
Tel: +61 2 9209 4169, Email: tech@speedx.com.au

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1. Product description

The **ResistancePlus™** MG Positive Control kit contains amplification controls for use with the **ResistancePlus™** MG kit. The positive control samples contain fragments of synthetic DNA for *Mycoplasma genitalium* MgPa and 23S rRNA targets, designed to simulate wild type *M. genitalium* or mutant *M. genitalium* with a 23S rRNA mutation (A2058G, A2059G, A2058T, A2058C, or A2059C *Escherichia coli* numbering).

2. Intended use

The **ResistancePlus™** MG Positive Control kit is intended to be used as positive amplification controls for the **ResistancePlus™** MG kit. This control is not intended for use with other assays or systems.

3. Kit contents

Number of tests: Sufficient for 10 tests (5 µl sample volumes)

Cap Colour	Contents #	Quantity
Neutral	MG, 23S rRNA wild type	1 x 50 µl
Green	MG, 23S rRNA A2058G	1 x 50 µl
Red	MG, 23S rRNA A2059G	1 x 50 µl
Blue	MG, 23S rRNA A2058T	1 x 50 µl
Yellow	MG, 23S rRNA A2058C	1 x 50 µl
Purple	MG, 23S rRNA A2059C	1 x 50 µl

Store contents separately from mastermix setup area

4. Shipping and storage

The components of the **ResistancePlus™** MG Positive Control kit are shipped on dry/blue ice. All components should be stored at -20°C upon receipt. Excessive freeze/thawing is not recommended.

When stored under the recommended conditions and handled correctly, activity of the kit is retained until the expiry date stated on the label. Do not use past expiry date.

5. Instructions for use

Please refer to the **ResistancePlus™** MG Positive Control IFU for detailed interpretation of results.

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