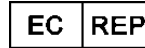
PlexPCR[®] Colour Compensation

For LightCycler[®] 480 Instrument I & II and cobas[®] z 480 analyzer

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SpeedX Pty Ltd, Suite G16 National Innovation Centre
Australian Technology Park, 4 Cornwallis Street, Eveleigh,
Sydney, NSW 2015, Australia
Tel: +61 2 9209 4169, Email: tech@speedx.com.au

FOR PROFESSIONAL USE ONLY*

*not for sale in the USA

1. Product description

The **PlexPCR[®] Colour Compensation (CC)** kit is for the generation of a **PlexPCR[®] Colour Compensation (CC)** file on the LightCycler[®] 480 Instrument I & II (LC480 I & LC480 II) and cobas[®] z 480 analyzer (z 480) (Roche). All **PlexPCR[®]** kits and **ResistancePlus[®]** kits use a specific combination of dyes, and the **PlexPCR[®] CC** file **MUST** be applied for correct analysis of results. Please refer to the **PlexPCR[®]** kit for intended use.

2. Kit contents

96-well plate: Sufficient for 2 colour compensation files for 20 µl reaction volume
384-well plate: Sufficient for 5 colour compensation files for 10 µl reaction volume

Table 1. Kit contents		
Cap Colour	Content	Quantity
Brown	488 [†] CC mix, 20x	1 x 15 µl
Green	510 [†] CC mix, 20x	1 x 15 µl
Yellow	580 [†] CC mix, 20x	1 x 15 µl
Purple	610 [†] CC mix, 20x	1 x 15 µl
White	640 [†] CC mix, 20x	1 x 15 µl
Black	660 [†] CC mix, 20x	1 x 15 µl
Orange	Plex Mastermix, 2x	1 x 1 ml
Blue	MgCl ₂ , 50 mM	1 x 1.2 ml
Neutral	CC template [#]	1 x 400 µl

[†] Emission channel on LC480 II (see Instructions for Use (IFU) for emission channel on LC480 I and z 480)

[#] Store template tubes separately from oligo mixes

3. Instructions

Please refer to the **PlexPCR[®] CC** kit Instructions for Use (IFU) for the detailed protocol (Ref No IF-IV0001). This document can be obtained by contacting tech@speedx.com.au.

4. Shipping and storage

The **PlexPCR[®] CC** kit is shipped on dry ice or ice gel packs. All components should be stored at -20°C upon receipt. Excessive freeze/thawing is not recommended.

When stored under the recommended conditions and handled correctly, activity of the kit is retained until the expiry date stated on the label. Do not use past expiry date.

5. Applying the PlexPCR[®] CC

The **PlexPCR[®] CC** file can be subsequently applied to all **PlexPCR[®]** kits or **ResistancePlus[®]** kits that have been created on the same instrument. Please refer to the **PlexPCR[®]** kits and the **ResistancePlus[®]** kits IFU for the details on how to apply the **PlexPCR[®] CC** file.

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