
ResistancePlus® GC

ResistancePlus® GC₍₅₅₀₎ for Applied Biosystems

7500 Fast and 7500 Fast Dx Instruments

2013025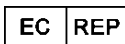

MT Promedt Consulting GmbH, Altenhofstrasse 80,
66386 St. Ingbert, Germany
Tel: +49 6894 581020, Email: info@mt-procons.com

SpeedX Pty Ltd, Suite G16 National Innovation Centre
Australian Technology Park, 4 Cornwallis Street, Eveleigh,
Sydney, NSW 2015, Australia
Tel: +61 2 9209 4169, Email: tech@speedx.com.au

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Not for sale in the USA

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1. Intended Use

The **ResistancePlus® GC** kit is a qualitative real-time PCR assay for the detection of *Neisseria gonorrhoeae* and the *gyrA* S91 wild-type and S91F mutant sequences as markers of susceptibility or resistance to ciprofloxacin. The **ResistancePlus® GC** kit can be used with the specimen types listed in the Instructions for Use (IFU).

The **ResistancePlus® GC** kit is intended to be used in professional settings such as hospitals, or reference or state laboratories. **It is not intended for self-testing, at home or point of care use.**

2. Kit contents

Sufficient for 25 tests (20 µl)

Cap Colour	Contents	Quantity
Blue	Plex Mastermix, 2x	1 x 250 µl
Brown	GC+gyrA Mix, 20x	1 x 25 µl
White	Control Mix 2, 20x	1 x 25 µl
Red	Internal Control Cells [#]	1 x 100 µl
Neutral	Nuclease Free Water	1 x 1 ml

[#] Store template tubes separately from oligonucleotide mixes, i.e. template or nucleic acid handling room.

3. Instructions and interpretation of results

Please refer to the **ResistancePlus® GC** IFU for detailed protocol and interpretation of results. This document can be obtained by contacting tech@speedx.com.au

4. Shipping and storage

The components of the **ResistancePlus® GC** kit are shipped on dry ice or ice gel packs. All components should be stored at -20°C upon receipt. It is recommended that freeze/thaw cycles are limited to 15.

When stored under the recommended conditions and handled correctly, activity of the kits is retained until the expiry date stated on the label. Do not use past expiry date.

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