
# ResistancePlus® MG S2A Positive Control

S2A-95004

**SpeedX Pty Ltd**  
Suite G16, National Innovation Centre  
Australian Technology Park, 4 Cornwallis Street, Eveleigh  
Sydney, NSW 2015, Australia  
Tel: +61 2 9209 4169, Email: [tech@speedx.com.au](mailto:tech@speedx.com.au)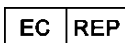

**MT Promedt Consulting GmbH**  
Altenhofstrasse 80  
66386 St. Ingbert, Germany  
Tel: +49 6894 581020, Email: [info@mt-procons.com](mailto:info@mt-procons.com)

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Not for sale in the USA

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## 1. Intended use

The **ResistancePlus®** MG S2A Positive Control kit contains amplification controls. The positive control samples contain plasmid DNA for *Mycoplasma genitalium* MgPa and 23S rRNA targets, designed to simulate wild type *M. genitalium* or mutant *M. genitalium* with a 23S rRNA mutation (A2058G, A2059G, A2058T or A2058C, *Escherichia coli* numbering).

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## 2. Kit contents

Sufficient for 2 tests per positive control (100 µl sample volume)

Cap colour	Contents <sup>#</sup>	Quantity
White	MG, 23S rRNA wild type	2 x 100 µl
Green	MG, 23S rRNA A2058G	2 x 100 µl
Orange	MG, 23S rRNA A2059G	2 x 100 µl
Blue	MG, 23S rRNA A2058T	2 x 100 µl
Yellow	MG, 23S rRNA A2058C	2 x 100 µl
Neutral	Dilution buffer	10 x 1 mL

<sup>#</sup> Store template tubes separately from oligo mixes, i.e. template or nucleic acid handling room

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## 3. Shipping and storage

The components of the **ResistancePlus®** MG S2A Positive Control kit are shipped on dry ice or ice gel packs. All components should be stored at -20°C upon receipt. It is recommended that freeze/thaw cycles are limited to less than 10.

When stored under the recommended conditions and handled correctly, activity of the kit is retained until the expiry date stated on the label. Do not use past expiry date.

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