PlexPCR® Colour Compensation

LightCycler® 480 Instrument I & II and cobas® z 480 analyzer

90001

MT Promedt Consulting GmbH

Altenhofstrasse 80

66386 St. Ingbert, Germany

Tel: +49 6894 581020, Email: info@mt-procons.com

SpeedX Pty Ltd

Suite G16, National Innovation Centre

Australian Technology Park, 4 Cornwallis Street, Eveleigh

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Not for sale in the USA

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1. Intended use

The **PlexPCR®** Colour Compensation (CC) kit is for the generation of a **PlexPCR®** CC file on the LightCycler® 480 Instrument I & II (LC480 I & LC480 II) and cobas® z 480 analyzer (z 480) (Roche). All **PlexPCR®** kits and **ResistancePlus®** kits use a specific combination of dyes, and the **PlexPCR®** CC file **MUST** be applied for correct analysis of results.

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2. Kit contents

96-well plate: Sufficient for 2 colour compensation files for 20 µl reaction volume

384-well plate: Sufficient for 5 colour compensation files for 10 µl reaction volume

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Store template tubes separately from oligo mixes, i.e. template or nucleic acid handling room

‡ Emission channel on LC480 II (see Instructions for Use (IFU) for emission channel on LC480 I and z 480)

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3. Instructions

Please refer to the **PlexPCR®** CC kit IFU for the detailed protocol (Ref No IF-IV0001). This document can be obtained by contacting tech@speedx.com.au.

The **PlexPCR®** CC file can be subsequently applied to all **PlexPCR®** or **ResistancePlus®** runfiles that have been created on the same instrument. Please refer to the **PlexPCR®** and **ResistancePlus®** kit IFU for the details on how to apply the **PlexPCR®** CC file.

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4. Shipping and storage

The **PlexPCR®** CC kit is shipped on dry ice or ice gel packs. All components should be stored at -20°C upon receipt.

When stored under the recommended conditions and handled correctly, activity of the kit is retained until the expiry date stated on the label. Do not use past expiry date.

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