# ResistancePlus® MG Positive Control

**REF** 95001

**U.S. Agent** SpeedX Inc, 701 Brazos Street, Ste 1616, TX, 78701 USA  
Tel: (512) 200 6918, Email: [tech@speedx.com.au](mailto:tech@speedx.com.au)

**EC REP** MT Promedt Consulting GmbH, Altenhofstrasse 80  
66386 St. Ingbert, Germany  
Tel: +49 6894 581020, Email: [info@mt-procons.com](mailto:info@mt-procons.com)

**SpeedX Pty Ltd**  
Suite G16, National Innovation Centre  
Australian Technology Park, 4 Cornwallis Street, Eveleigh  
Sydney, NSW 2015, Australia  
Tel: +61 2 9209 4169, Email: [tech@speedx.com.au](mailto:tech@speedx.com.au)

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## 1. Intended use

The **ResistancePlus®** MG Positive Control kit contains amplification controls. The positive control samples contain plasmid DNA for *Mycoplasma genitalium* MgPa and 23S rRNA targets, designed to simulate wild type *M. genitalium* or mutant *M. genitalium* with a 23S rRNA mutation (A2058G, A2059G, A2058T, A2058C, or A2059C *Escherichia coli* numbering).

## 2. Kit contents

Sufficient for 10 tests (5 µl sample volumes)

Cap colour	Contents <sup>#</sup>	Quantity
Neutral	MG, 23S rRNA wild type	1 x 50 µl
Green	MG, 23S rRNA A2058G	1 x 50 µl
Red	MG, 23S rRNA A2059G	1 x 50 µl
Blue	MG, 23S rRNA A2058T	1 x 50 µl
Yellow	MG, 23S rRNA A2058C	1 x 50 µl
Purple	MG, 23S rRNA A2059C	1 x 50 µl

<sup>#</sup> Store contents separately from mastermix setup area

## 3. Shipping and storage

The components of the **ResistancePlus®** MG Positive Control kit are shipped on dry ice or ice gel packs. All components should be stored at -20°C upon receipt. It is recommended that freeze/thaw cycles are limited to less than 10.

When stored under the recommended conditions and handled correctly, activity of the kit is retained until the expiry date stated on the label. Do not use past expiry date.

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