# ResistancePlus<sup>®</sup> GC

## ResistancePlus<sup>®</sup> GC<sub>(675)</sub> for Bio-Rad CFX96<sup>™</sup> IVD and CFX96 Touch<sup>™</sup>

**REF** 2015001

EN

### FOR PROFESSIONAL USE ONLY

Not for sale in the USA

#### 1. Intended use

The **ResistancePlus<sup>®</sup> GC** kit is a qualitative real-time PCR assay for the detection of *Neisseria gonorrhoeae* and the *gyrA* S91 wild-type and S91F mutant sequences as markers of susceptibility or resistance to ciprofloxacin. The **ResistancePlus<sup>®</sup> GC** kit can be used with the specimen types listed in the Instructions for Use (IFU).

The **ResistancePlus<sup>®</sup> GC** kit is intended to be used in professional settings such as hospitals, or reference or state laboratories. **It is not intended for self-testing, at home or point of care use.**

#### 2. Kit contents

Sufficient for 100 tests (20 µl)

Cap colour	Contents	Quantity
Blue	<b>Plex</b> Mastermix, 2x	1 x 1 ml
Brown	GC+gyrA Mix, 20x	1 x 100 µl
White	Control Mix 3, 20x	1 x 100 µl
Red	Internal Control Cells <sup>#</sup>	1 x 500 µl
Neutral	Nuclease Free Water	1 x 1 ml

<sup>#</sup> Store template tubes separately from oligo mixes, i.e. template or nucleic acid handling room

#### 3. Instructions

Please refer to the **ResistancePlus<sup>®</sup> GC** IFU for the detailed protocol (Ref No IF-IV0009). This document can be obtained by contacting [tech@speedx.com.au](mailto:tech@speedx.com.au).

#### 4. Shipping and storage

The components of the **ResistancePlus<sup>®</sup> GC** kit are shipped on dry ice or ice gel packs. All components should be stored at -25°C to -15°C upon receipt. It is recommended that freeze/thaw cycles are limited to 15.

When stored under the recommended conditions and handled correctly, activity of the kit is retained until the expiry date stated on the label. Do not use past expiry date.

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