ResistancePlus® GC Control

REF 95003

EN

FOR PROFESSIONAL USE ONLY

1. Intended use

The control reagent(s) consist of plasmid or synthetic DNA carrying target sequence(s) intended as a general-purpose non-assay specific reagent, for use as an amplification control in any qPCR reaction. This reagent does not contain any pathogen(s) and we do not make any claims, such as an assigned quantity (value) or specific quality parameters. This reagent has no diagnostic value and is provided as an accessory reagent, which possess no critical performance characteristics

2. Description

The **ResistancePlus®** GC Control kit contains amplification controls. The positive control samples contain synthetic DNA for *Neisseria gonorrhoeae porA*, *opa*, and *gyrA* S91 wildtype or *gyrA* S91F mutant targets. The Negative Control is for use as a negative control in PCR.

3. Kit contents

Cap colour	Contents*	Quantity
White	GC, <i>gyrA</i> wild type	1 x 100 µl
Green	GC, <i>gyrA</i> S91F mutant	1 x 100 µl
Black	Negative Control	1 x 100 µl

Store template tubes separately from oligo mixes, i.e. template or nucleic acid handling room

4. Shipping and storage

The components of the **ResistancePlus®** GC Control kit are shipped on dry ice or ice gel packs. All components should be stored at -25°C to -15°C upon receipt. It is recommended that freeze/thaw cycles are limited to less than 10.

When stored under the recommended conditions and handled correctly, activity of the kit is retained until the expiry date stated on the label. Do not use past expiry date.

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