ResistancePlus[®] MG FleXible

ResistancePlus[®] MG FleXible for GeneXpert[®] Instrument Systems

REF

S2A-2000410

EN

FOR PROFESSIONAL USE ONLY

Not for sale in the USA

1. Intended use

The **ResistancePlus[®] MG FleXible** kit is a qualitative multiplexed *in vitro* diagnostic real-time PCR test for the identification of *Mycoplasma genitalium* and detection mutations in the 23S rRNA gene (A2058G, A2059G, A2058T and A2058C, *Escherichia coli* numbering), that are associated with resistance to azithromycin (macrolide antibiotic). The **ResistancePlus[®] MG FleXible** kit can be used with the specimen types listed in the Instructions for Use.

2. Kit contents

Number of tests: 10 reactions

Box no.	Part no.	Cap colour	Contents	Quantity
1	2000410-R	Blue	Plex Mastermix, 2x	1 x 440 µl
		Brown	MG+23S Mix, 20x	1 x 50 µl
		Red	Internal Control Cells*	1 x 100 µl
		N/A	ResistancePlus[®] MG FleXible labels*	10 labels
		N/A	MG FleXible Mix label	1 label
2	2000410-CART	N/A	ResistancePlus[®] MG FleXible Cartridge	10 cartridges

Store template tubes separately from oligo mixes, i.e. template or nucleic acid handling room

* **Do not dispose of cartridge labels**

3. Instructions

Please refer to the **ResistancePlus[®] MG FleXible** kit IFU for the detailed protocol and how to import the ADF (Ref No IF-IV0012). This document can also be obtained on the **ResistancePlus[®] MG FleXible** CD (P/N:CD-MG-FLEX) which is provided with each shipment.

4. Shipping and storage

The components of the **ResistancePlus[®] MG FleXible** kit (P/N:2000410-R) are shipped on dry ice or ice gel packs and stored at -25°C to -15°C. **ResistancePlus[®] MG FleXible** Cartridges (P/N:2000410-CART) are shipped and stored 2°C – 28°C. When stored under the recommended conditions and handled correctly, activity of the kit is retained until the expiry date stated on the label. Do not use past expiry date.

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