ResistancePlus® MG Positive Control

REF

95001

EN

FOR PROFESSIONAL USE ONLY

1. Intended use

The control reagent(s) consist of plasmid or synthetic DNA carrying target sequence(s) intended as a general-purpose non-assay specific reagent, for use as an amplification control in any qPCR reaction. This reagent does not contain any pathogen(s) and we do not make any claims, such as an assigned quantity (value) or specific quality parameters. This reagent has no diagnostic value and is provided as an accessory reagent, which possess no critical performance characteristics

2. Description

The **ResistancePlus®** MG Positive Control kit contains amplification controls. The positive control samples contain plasmid DNA for *Mycoplasma genitalium* MgPa and 23S rRNA targets, designed to simulate wild type *M. genitalium* or mutant *M. genitalium* with a 23S rRNA mutation (A2058G, A2059G, A2058T, A2058C, or A2059C *Escherichia coli* numbering).

3. Kit contents

Sufficient for 10 tests (5 µl sample volumes)

Cap colour	Contents*	Quantity
Neutral	MG, 23S rRNA wild type	1 x 50 µl
Green	MG, 23S rRNA A2058G	1 x 50 µl
Red	MG, 23S rRNA A2059G	1 x 50 µl
Blue	MG, 23S rRNA A2058T	1 x 50 µl
Yellow	MG, 23S rRNA A2058C	1 x 50 µl
Purple	MG, 23S rRNA A2059C	1 x 50 µl

* Store template tubes separately from oligo mixes, i.e. template or nucleic acid handling room

4. Shipping and storage

The components of the **ResistancePlus®** MG Positive Control kit are shipped on dry ice or ice gel packs. All components should be stored at -25°C to -15°C upon receipt. It is recommended that freeze/thaw cycles are limited to less than 10. When stored under the recommended conditions and handled correctly, activity of the kit is retained until the expiry date stated on the label. Do not use past expiry date.

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